cálido confort

## Flexyheat Air

Almohadilla térmica  
Heating Pad  
Almofadinha térmica  
Coussin chauffant

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## ENGLISH

### WARNING

**Please read the instructions for use carefully prior to using the product. Store these in a safe place for future reference.**

Prior to plugging in the product, check that your mains voltage is the same as the one indicated on the product label.

The mains connection cable must not be tangled or wrapped around the product during use.

Do not pull on the cable to unplug it or use it to carry the product.

Do not wrap the cable around the control.

Do not cover or lay the control on top of the product during use in order to ensure its proper operation.

Do not cover the product with any heavy or thick objects and do not place on top of any other heating elements in order to ensure the proper operation and use of the product.

Use without folding.

Do not stick any pins or needles or any other pointy metal objects into the product.

In the event that any of the parts of the product become damp or wet, do not try to use it until it is dry.

Proceed according to the conservation section of this manual for cleaning.

Children under the age of three should not use this appliance due to their inability to react to overheating

The product must not be used by children unless the controls have been preset by a parent or guardian, or the child has been at least properly instructed on how to use the controls in a secure manner.

This product should not be used by people insensitive to heat and other highly vulnerable people who are unable to react to overheating

Do not use the product to warm up animals.

Despite the reduction in temperature and the automatic disconnection provided for in the product, the temperature controls should be set to the minimum level of operation (level 1) in the event of continued use.

The product should only be inflated with air. No other gases such as helium or nitrogen or the use of liquids for this purpose may be used under any circumstances.

The product should be inflated prior to being used, when it is not hot.

The product should be inflated by blowing it up either by respiratory means or using manual inflators. We do not recommend the use of compressors for the inflation of the pad due to the fact that this can

increase the internal pressure sharply increasing the possibility of it bursting.

This product is not intended for medical use in hospitals.

The product you purchased may only be used with the control supplied with the product as indicated on the product label.

Materials will suffer wear and tear over time and after prolonged use.

Carry out a regular inspection of the product including the control and flexible cable to check for any signs of wear or damage.

Unplug the product immediately from the mains in the event of any breakdown and contact an authorised Technical support service.

In order to prevent any risk of danger, do not open the control or connection. Only qualified technical personnel from the brand's Official Technical support service may carry out repairs or procedures on the device.

Any incorrect use or improper handling of the product shall render the warranty null and void. Only an authorised technical support service centre may carry out repairs on this product.

**B&B TRENDS, S.L.** disclaims all liability for damages that may occur to people, animals or objects, for the non-observance of these warnings.

## INSTRUCTIONS FOR USE

This product can be used as a conventional pad (flat) or as a cushion (inflating the product according to your preferences). The level of cushioning of the product can be adjusted according to how much it is inflated when used as a cushion. Adjust according to your preferences and/or needs while always bearing in mind the safety instructions contained in this instruction manual. Do not fully inflate.

The pad should be inflated by pinching slightly with your fingers on the base of the inflation valve until you feel the amount of air that is being entered into the pad increasing. This technique must be applied to release the retention valve and let the air circulate.

The product should be deflated with the valve nozzle open while pinching slightly with your fingers on the base of the valve (as well as during inflation), putting pressure on the surface of the pad until all the air has been removed from its interior.

Prior to connecting the product to the mains, check your power supply voltage to ensure it is the same as the one indicated on the label included on the product. Connect the product to the appropriate mains.

In order to turn on the product, you must select the slider switch on the control, setting it to position 1, 2 or 3 (Fig.1).

From this time on, the product is operational.

The maximum operation time is 90 minutes, after which the automatic safety shut-down will immediately activate. The product will no longer heat up when in this state.

Select the "0" position in order to turn off the product before you disconnect the device from the mains.

## ENGLISH

### TROUBLESHOOTING GUIDE:

Possible operational problems

PROBLEM	RECOMMENDATION
<b>The regulator control does not light up when the switch is turned on.</b>	Ensure that the regulator is properly connected to the product and that there is voltage in the socket of the mains used and that the product is properly plugged in.
<b>After a certain operation time, the product temperature seems to reduce.</b>	The product has an auto-stop system that activates after 90 minutes of continuous operation. After this time, the light on the regulator starts to flash. If you want to continue to use it, change the temperature level to level 0, wait a few seconds, and then select the desired temperature.
<b>I cannot inflate the pad or it is very difficult to do so.</b>	Ensure that you pinch the base of the inflation valve with your fingers to allow air to enter. If you notice that the air is entering the pad, but is coming out, there may be a leak. Contact Daga's technical service if this occurs.
<b>The product has an air leak.</b>	Contact Daga's technical service.

### PRODUCT DISPOSAL AND CONSERVATION

#### PRODUCT CONSERVATION

Deflate the product prior to storage.

Unplug the product when you do not intend to use it for an extended period of time to prolong its shelf-life. Store it in its original packaging in a dry place, trying to fold it as little as possible and preventing the formation of creases.

Do not cover the product with other heated elements during its operation. Let it cool down prior to storage. Do not place any objects on top of the product during storage in order to prevent the formation of creases.

#### PRODUCT CLEANING

If you are machine-washing the product, ensure that the inflation valve is fully closed to prevent water entering its interior.

Unplug and remove the regulator prior to cleaning the product.

The entire product, except for the control, can be washed by hand or in the machine, using liquid soap at a temperature not exceeding 30 °C and selecting a program for washing delicate garments.

ENGLISH

## PRODUCT DISPOSAL

Do not dispose of the product at the end of its shelf-life. Contact the organic collection centre closest to your home in accordance with the waste electrical and electronic equipment (WEEE) Directive 2012/19/EU to contribute to the protection of the environment. Request detailed information regarding their current methods of disposing of packaging and scrappage of used devices from your distributor.

We hope that you will be satisfied with this product.

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**This product complies with European Directive 2012/19/EU on electrical and electronic devices, known as WEEE (Waste Electrical and Electronic Equipment), which provides the legal framework applicable in the European Union for the disposal and reuse of waste electronic and electrical devices.**

**Do not dispose of this product in the bin, instead going to the electrical and electronic waste collection centre closest to your home.**

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## WARRANTY REPORT

**B&B TRENDS, SL.** guarantees compliance of this product for the purpose for which it is intended for a period of two years for the entire Spanish territory. In the case of breakdown during the term of this warranty, you are entitled to repair or to the replacement of the product at no charge if the former is unavailable, unless one of these options prove impossible or disproportionate. In this case, you can then choose a reduction in price or cancellation of the sale, which must be dealt with directly with the sales vendor. This also covers replacement of spare parts provided that the product has been used according to the recommendations specified in this manual for both cases, and has not been tampered with by any third party that is unaccredited by **B&B TRENDS, SL.** This warranty does not affect your rights as a consumer in accordance with the provisions of Law 23/2003 dated 10 July 2003 on warranties on the sale of consumer goods in the Spanish territory.

## USE OF WARRANTY

Consumers should contact a technical service centre authorised by **B&B TRENDS, SL.** for repair of the product as any tampering thereof by persons not authorised by **B&B TRENDS, SL.**, as well as misuse or inappropriate use, will lead to the cancellation of this warranty. The warranty must be fully completed and delivered along with the receipt or delivery docket for the effective exercise of rights under this warranty.

Claims must take place within three years from delivery of the product according to the provisions of the warranty, and the consumer must inform the sales vendor of the lack of conformity, provided that such claim is made within the statutory period of warranty. This warranty should be retained by the user as well as the invoice, receipt or the delivery docket to facilitate the exercise of these rights. For technical service and after-sales care outside the Spanish territory, please submit your query to the point of sale where you purchased the item or by contacting **B&B TRENDS, SL.**

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