cálido confort

AR-600

Aparato para el tratamiento de alergias  
Allergy reliever treatment device  
Aparelho para o tratamento de alergias  
Appareil pour le traitement des allergies

calidad

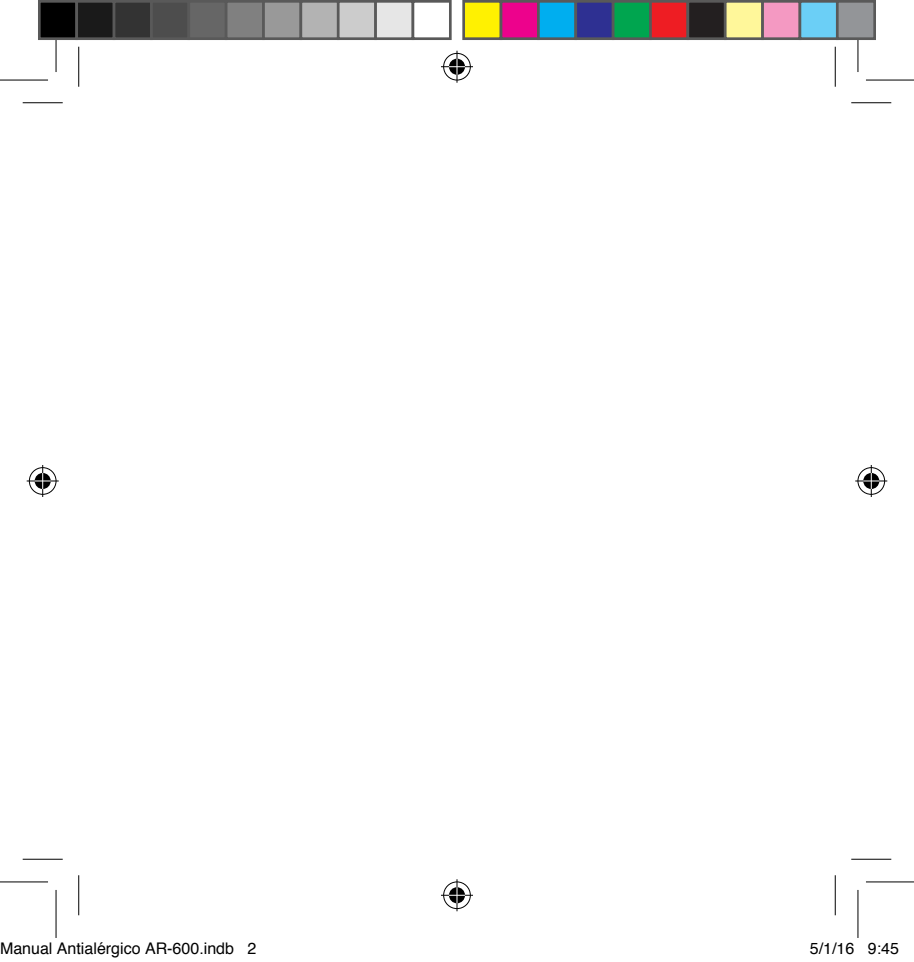
control

seguridad

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## IMPORTANT WARNING

ENGLISH

**Carefully read the instructions before using the product keep them in a safe place for later reference.**

## INTRODUCTION

In first place we would like to personally thank the Daga team for acquiring this product. We trust that it is totally satisfactory. We hope that you continue to put your faith in us and do not hesitate to contact Daga customer service for any doubt, clarification or suggestion.

The Daga AR-600 team has been designed specifically to alleviate hay fever reducing the symptoms of millions of people all over the world mainly during months with high pollen concentration in the atmosphere. The reaction of the human body when it comes into contact with an allergen is to produce antibodies that release histamine in an attempt to fight against external agents that it considered to be malign. In reality these elements that cause the allergy are not harmful to the human body

(pollen, mites, animal hairs, among other things) but they are indeed interpreted as such by the immune systems of some individuals.

This device uses a dual phototherapeutic treatment that consists of applying infrared light of different wavelengths (652nm and 940nm) inside the nasal cavities. Through applying visible infrared light (652nm) we are able to eliminate the cells that release histamine, thus reducing nasal inflammation and congestion for the individual. On the other hand, thanks to invisible infrared light (940nm) calorific energy is released inside the nasal cavities causing the blood vessels to dilate, assisting circulation.

## SAFETY MEASURES

### Contraindications:

In no way should this device be used as a substitute for medical treatment. Children should be supervised by an adult to avoid using the device as a toy.

This device is not designed to be used by persons (including children) with reduced physical, sensory or psychological capacities, or who lack the necessary experience and knowledge, unless they are supervised by a person responsible for their safety.

**Avoid using the device in the following cases:**

- While you are asleep.
- On animals.
- While driving or using heavy or dangerous machinery.
- After ingesting substances that alter your perception (e.g. analgesic

medicine, alcohol, drugs).

- While carrying out activities that could have an unpredictable reaction, becoming dangerous.
- If you suffer from nasal polyps, hypersensitivity or circulatory problems in the nasal area.

When in any doubt over whether the device is appropriate for you, consult your doctor. **Consult your doctor before** using the device if:

- You have a pacemaker or electronic implants.
- You suffer from hypertension.
- You suffer from epileptic fits.

## Use:

- Only use the device for the purposes indicated in this instruction manual.
- Do not stare into the infrared beams of light emitted by the device. It could harm your vision.
- Do not insert the infrared light probe into areas other than the nasal cavities.
- Always use the device with the cables provided. Do not introduce any other element to the device.
- It is not permitted to use this device while connecting yourself to high frequency surgical equipment.
- Do not use the device around shortwave or microwave equipment.
- Stop using the device if you feel nausea, dizziness, fever, muscular pain or any other type of discomfort. Consult your doctor if this happens.

- 
- 
- Do not use the device in water or around excessively humid surroundings such as saunas, showers or Jacuzzis.
  - Do not share the device with another person that could have an infectious disease. This will avoid possible contagion. You can obtain an additional probe for another person through the technical assistance service.

## Repair:

Only specialised personnel should perform repairs to this electrical appliance. Any incorrectly performed repair can lead considerable hazards for the user. Always contact the after-sales service for repair authorised by B&B Trends. Failure to observe these instructions will lead to the total loss of the product warranty.

**B&B TRENDS SL** accepts no liability for any damage that may be caused to people, animals or objects owing to the failure to observe this warning correctly.

## GENERAL INFORMATION:

First check that the appliance is complete and undamaged. Contact the store where you purchased the product in the event of any doubt.

Through you purchase you have acquired:

- 1 Main Daga AR-600 anti-allergic treatment equipment.
- 1 Infrared emission probe for nasal cavities.

\* Batteries not included.

## Inserting the batteries:

To use this product you should insert 2 AA batteries (not included) into the device. To do this, remove the cover located on the lower part of the device and insert the batteries, making sure to fit the correct poles, as indicated on the inside of the compartment. When the batteries are about the run out, the on/off button will flash intermittently. The indicators that the batteries must be replaced to be able to continue using the device. Always replace all the batteries at once, trying to avoid mixing new batteries with old ones so that they do not run out prematurely.

## INSTRUCTIONS FOR USE

Before beginning to use the product, make sure that there is no apparent damage to its parts. Make sure that the probes to be introduced to the nasal cavities emit red light when they are in use. In case there is any damage to the device, please contact our aftersales service or one of the authorised distributors.

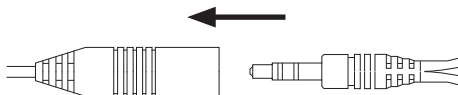
**IMPORTANT:** Under no circumstance should the device substitute any medical treatment, as it alleviates allergy symptoms. Results can vary from person to person, and they are neither immediate nor definitive. Follow the instructions that are presented below in order to start up the appliance:

1. Remove the product from its original packaging taking special care not to damage any of its parts. Do not use excessive force to remove the product from the box.

- 
- 
2. Insert the batteries into the device if you have not already done it.
  3. Interconnect the two parts of the equipment by joining the male connector Jack of the nasal probe with the female connector of the central equipment.

- 
- 
4. Insert the nasal probe appropriately and comfortably inside your nose. Use it in pincer mode to facilitate introducing it into the area to be treated. If you have nasal congestion remove the maximum amount of mucus possible from the nasal cavity before applying the treatment in order to increase its effectiveness.
  5. Press the on/off button on the upper face of the central equipment to start the treatment.
  6. Wait for approximately 3 minutes, which is the established time for this treatment. The device will turn off automatically after this time, then sounding a noise. Even so, should you wish, you can finish before the 3 minutes have passed by pressing the on/off button.
  7. It is recommended to use this device approximately 3 or 4 times a day leaving 5 to 6 hours between sessions.
  8. Once the session has finished make sure the the device is turned off completely and disconnect the two parts of the equipment. If the device is not going to be used for a long time, take the batteries out of the device and keep them in a safe place.

9. Clean the parts inserted into the nose well after each use. Consult the product cleaning section to find out how to do it.

## TECHNICAL DATA

Type of treatment	Dual infrared phototherapy
Emitting mode	Continuous
Time of treatment	3 minutes
Wavelengths used	652 nm (visible light) and 940 nm (invisible light)
Dimensions	86mm x 60mm x 38mm
Weight	80g (without batteries)
Power	2 AAA batteries
Atmospheric conditions for use	15°C – 40°C (59°F – 104°F) Humidity 30% – 85% Pressure 80 kPa – 105 kPa
Atmospheric conditions for storage	-20 °C – 60 °C (-4°F – 140°F) Humidity 30% – 85% Pressure 80 kPa – 105 kPa

## PRODUCT CONSERVATION

If you do not plan to use the appliance for an extended period of time, you should store it in its original packaging in a clean, cool and dry area. Always take the batteries out of the device when it is not going to be used for a long time.

## PRODUCT CLEANING

This product can only be cleaned by hand, using a soft cloth slightly moistened with a mild soap. It is recommended to clean the parts inserted into the nose well after each use. The use of abrasive cleaning products or brushes, solvents, benzine, thinner or alcohol is completely prohibited. Never immerse the appliance or any of its parts in water or excessively moisten it in order to clean it. Do not re-use the appliance until it is completely dry.

## PRODUCT DISPOSAL

Do not dispose of the product at the end of its shelf-life. Contact the organic collection centre closest to your home in accordance with the waste electrical and electronic equipment (WEEE) Directive 2012/19/EU to contribute to the protection of the environment. Request detailed information regarding their current methods of disposing of packaging and scrappage of used devices from your distributor.

We hope that you will be satisfied with this product.

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**Carefully read the instructions for use.**

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**The product may contain batteries. Remove them before disposing of the product and dispose in special containers approved for this purpose.**

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**The EC marking certifies that complies with directive 93/42/EEC.**

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**Fabricant.**

Representante autorizado en la UE.

**Authorized Representative in the EU.**

Representante autorizado na UE.

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**B&B Trends, S.L.**

C/Catalunya, 24 P.I. Ca N'Oller. 08130 Santa Perpetua de Mo-  
guda. Barcelona – España

**Andon Health Co., Ltd.**

Nº3 and 10 Jinping Street, YaAn Road, Nankai District, Tianjin  
300190, China.

**Lotus Global Co., Ltd.**

1 Four Seasons Terrace West Drayton, Middlesex London,  
UB79GG, United Kingdom.

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### ENGLISH / WARRANTY REPORT

**B&B TRENDS, SL.** guarantees compliance of this product for the purpose for which it is intended for a period of two years the entire Spanish territory. In the case of breakdown during the term of this warranty, you are entitled to repair or to the replacement of the product at no charge if the former is unavailable, unless one of these options prove impossible or disproportionate. In this case, you can then choose a reduction in price or cancellation of the sale, which must be dealt with directly with the sales vendor. This also covers replacement of spare parts provided that the product has been used according to the recommendations specified in this manual for both cases, and has not been tampered with by any third party that is unaccredited by **B&B TRENDS, SL.** This warranty does not affect your rights as a consumer in accordance with the provisions of Law 23/2003 dated 10 July 2003 on warranties on the sale of consumer goods in the Spanish territory.

### USE OF WARRANTY

Consumers should contact a technical service centre authorized by **B&B TRENDS, SL.** for repair of the product as any tampering thereof by persons not authorized by Daga, as well as misuse or inappropriate use, will lead to the cancellation of this warranty. The warranty must be fully completed and delivered along with the receipt or delivery docket for the effective exercise of rights under this warranty. Claims **must take place within 2 years** from deliver of the project according to the provisions of the warranty, and the consumer must inform the sales vendor of the lack of conformity, provided that such claim is made within the statutory period of warranty. This warranty, as well as the invoice, receipt, or the delivery order must be retained by the user to facilitate the exercise of these rights. For technical service and after-sales care outside the Spanish territory, please submit your query to the point of sale where you purchased the item or by contacting **B&B TRENDS, SL.**

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JULEN S.A.T
ELECTRICIDAD RINCON, S.A. - S.A.T.